

SAFETY-FLOW™ Instructions for Use

OUTLET DEVICE




1. To Empty Drainage Bag: Hang the bag so that the Outlet Device Arrow is pointing straight down.

2. To Open Device: Place thumb at the top of the device and finger on the green lever.


3. Lift the lever up, following the arrow, to the open position.

4. Drain urine.

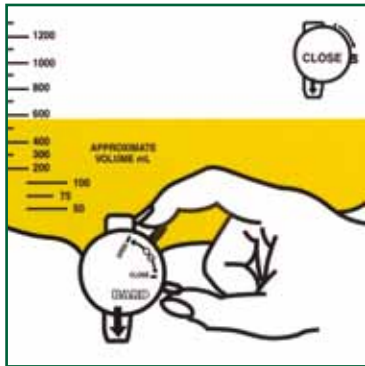

CLICK TO CLOSE

5. To Close Device: Place thumb on the support below the closed position and finger on the green lever.

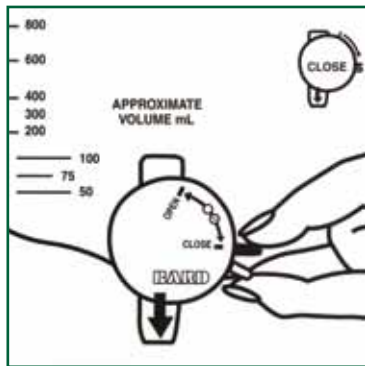

6. Move the lever down, following the arrow, to closed position.

LISTEN FOR THE CLICK TO CONFIRM THAT THE DEVICE IS CLOSED.

7. Dispose of the collected urine according to hospital protocol.

Bard Medical Division
 C. R. Bard, Inc.
 8195 Industrial Blvd.
 Covington, GA 30014
 800.526.4455
 www.bardmedical.com

Bard and Safety-Flow are trademarks and/or registered trademarks of C. R. Bard, Inc. Please consult product inserts and labels for any indications, contraindications, hazards, warnings, cautions and directions for use.
 © 2010 C. R. Bard, Inc. All rights reserved.
 1008-19 R08/10 XXX PXX/10 XM